


# Hollister Feeding Tube Attachment Device


## Application and Removal

### Preparing the skin:


Wash patient's nose and cheeks with a mild soap and water. Rinse thoroughly and dry completely. Do not use oils and lotions or emollients as the oily residue will interfere with the adhesion of the barrier.

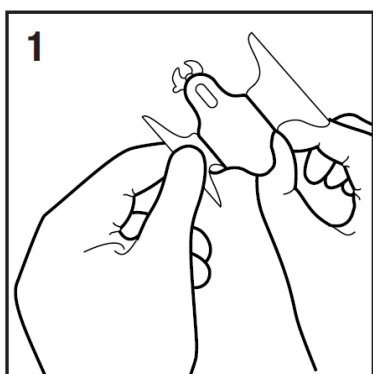

1. Remove the release paper from the adhesive barrier.

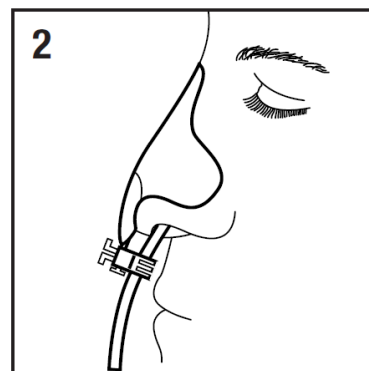

2. Position the device so the clamp extends approximately 13mm below the tip of the nose. Apply the vertical centre of the barrier to the bridge of the patient's nose, and press it down firmly, but gently, from the centre to the edges to ensure proper adhesion.

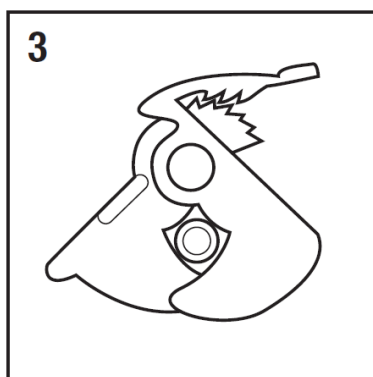


3. Secure the feeding tube by closing the arms of the device around it and squeeze the arms firmly to fully engage the serrated teeth.

**Note:** The feeding Tube Attachment Device should be monitored regularly to confirm barrier adhesion to the nose.

### For more information

**Australia Freecall: 1800 335 911**

**New Zealand Freecall: 0800 167 866**

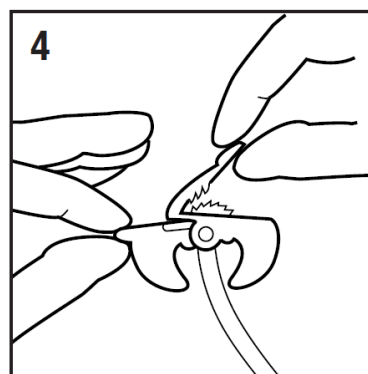

4. To unlatch, lift the release lever away from the serrated teeth. Place fingertip at the ridge on the opposite arm and rotate arm to open. Remove the device from the feeding tube, starting at the edge, peel the adhesive barrier from the patient's nose. Discard device into appropriate receptacle.



**CAUTION:** The Hollister Feeding Tube Attachment Device was designed as an alternative to tape. It does not eliminate any of the risks associated with the use of feeding tubes. For proper use of feeding tube, be sure to consult the feeding tube product instructions. When using the Feeding Tube Attachment Device, be sure to follow current standard operating procedure for the placement, monitoring